


Instructions to mount Dual Xenon Lightsource to Swivel Hanger

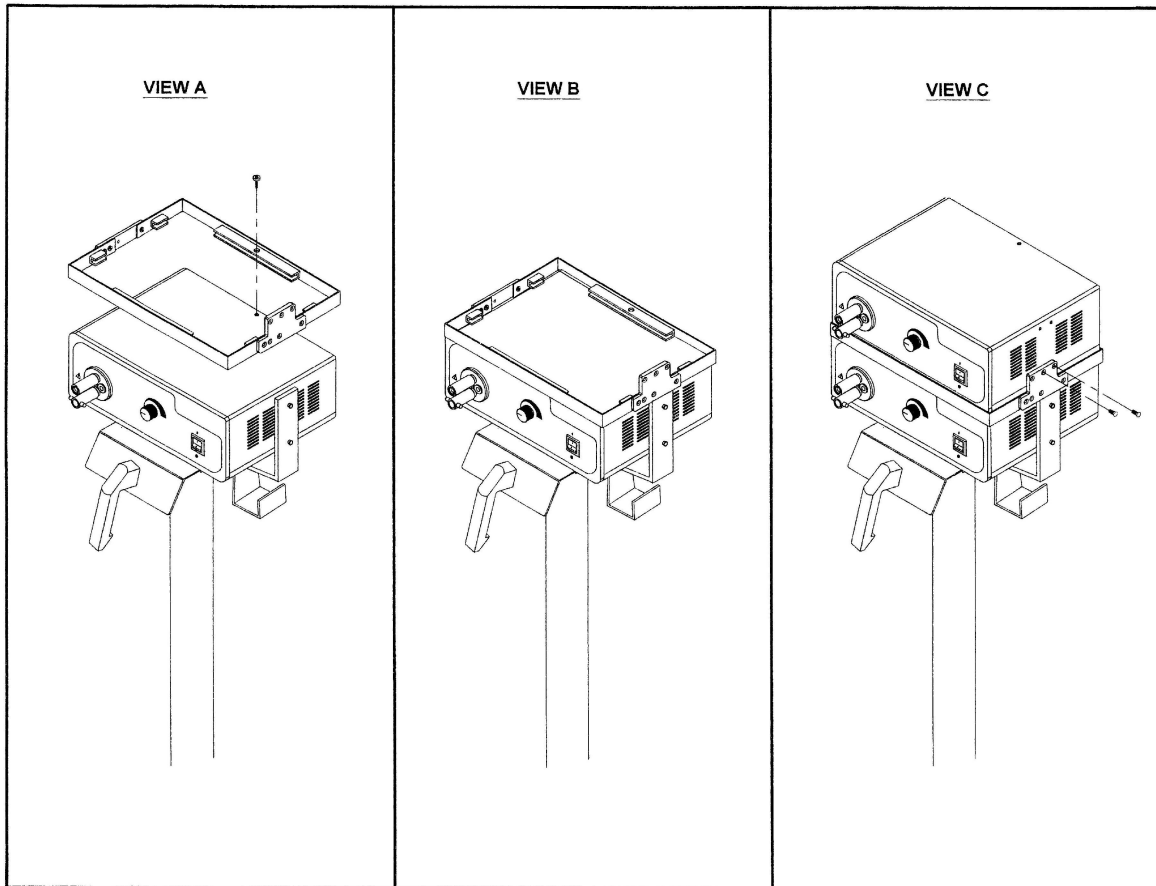

- Step 1. After the SSL0033 Sunoptic floorstand is assembled, mount a lightsource onto the swivel hanger.
- Step 2. Remove the top screw on mounted unit and place the stack tray onto the unit. Secure it with the 6-32 socket head screw provided. (See View A & B)
- Step 3. Remove the key hole bracket on the side of the second lightsource unit and place it on top of the tray.
- Step 4. Secure the top unit to the stack tray using the two (2) 8-32 flathead screws provided. (See View C)
- Step 5. Install the dual power cord female connector into each male receptacle on the back of the units.