


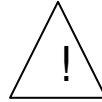

WARNING



BEFORE PLACING THIS UNIT IN SERVICE YOU MUST PERFORM THE FOLLOWING PROCEDURE

Date and Time Setting Procedure

1. Remove unit from packaging, connect to a monitor and attach a key board.
2. Power up unit by pressing the power button; during this time press and hold the 'F2' key. BIOS setup utility screen will appear.
3. Use the 'Down Arrow Key' to highlight 'System time'. Hour will be highlighted.
4. Use the +/- to make the changes to the hour setting.
5. Use the 'Tab' key to highlight the minutes and with the +/- key, change the minute setting.
6. Use the "Down arrow" key to change the month.
7. Use the 'Tab' key to highlight the year and with the +/- key, change the day.
8. Use the 'Tab' key to highlight the year and with the +/- key, change the year.
9. Press 'F10' key to save and exit.
10. Continue with system start up. Date and time are now set to your local settings.
11. Shut down the system with front panel button.
12. Disconnect monitor and keyboard.
13. Unit is ready to be placed into service.