




## Installation of Camera Bracket onto Floorstand


### STEP 1

Remove handle from floorstand base using the 5mm hex allen key provided to remove the two screws securing the handle from the backside of the flange.

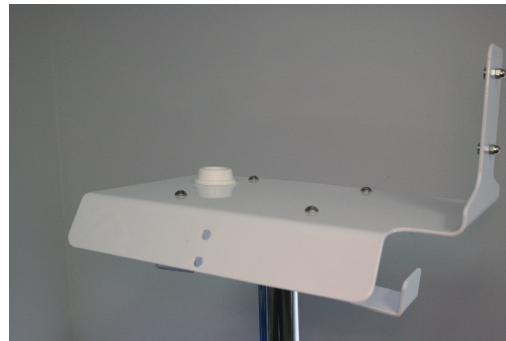

### STEP 2

Place camera bracket onto bent section of the floorstand and reinstall handle with the two screws.

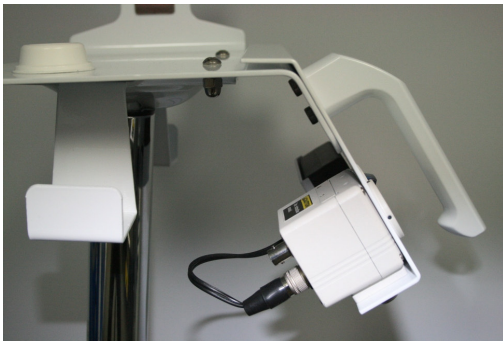

**STEP 3**

Plug power cord into power supply and S-Video cable into back of the CCU.



**STEP 4**

Place TITAN Lightsource onto floorstand and secure with cotter pin. Headlight Camera is now ready to be plugged into CCU on front of camera bracket. Make sure arrow on label on the camera connector is at the 12 O'clock position while inserting into CCU. The fiber-optic cable can be plugged into Wolf port on lightsource.



**STEP 5**

Plug power cord from Lightsource and CCU power supply into the wall outlets. Plug S-Video cable to the monitor that will be used to show the video. Turn on switch for the camera bracket and the lightsource and system is ready for use.