



## **OPTICAL CLEANING KIT INSTRUCTIONS FOR USE**

### Cleaning Kit Contents

- 2 cotton-tipped swabs
- 7.5 ml bottle of denatured alcohol

### Instructions

1. Use compressed air to remove surface dust without directly contacting the optic with the spray tip.
2. Soak one end of a cotton tipped swab with the solvent.
3. Using a circular motion starting from the edge of the optic moving toward the center, wipe away any dust or dirt.
4. Repeat if necessary using the opposite end of the cotton swab.
5. Allow 10 minutes for the solvent to dry before using
6. Dispose of cotton tipped swab after usage.

LIT-206

Rev.  
Date of revision:08/28/14  
Page 1 of 2



**Warnings:**

1. **Wear latex gloves and proper clothing before handling the solvent.**
2. **Do not apply solvent to a warm light source.**
3. **Do not mix solvent with other chemicals.**
4. **Avoid sources of ignition. Do not use around open flames.**
5. **In case of ingestion contact a poison control center immediately.**

LIT-206

Rev.  
Date of revision:08/28/14  
Page 2 of 2